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GB

Warning!

These instructions must be studied carefully before commissioning. Claims under warranty will be invalidated in the event of damage caused by non-compliance with the installation instructions. No liability is accepted for any resulting consequential loss.

General information

The call module serves for initiating calls from the patient bed. Connection option for pear / multiple push-buttons and diagnostic device with alarm contact as well as a microphone connection unit. Individual configuration of the bed ID and connection of up to 8 call modules to the System 99 bed bus. The various feature variants can be found in Table 1.

Mounting

Fasten the call module to the flush-mounted box with suitable screws as shown in Fig. 1. The module can also be mounted with the optionally available spread clamp set (Part No. 73073X) as described in Fig. 2.

Installation

All assembly and installation work may be carried out only when the device is disconnected from power. The call button is connected to the room terminal (Part No. 76921B1) or a room electronic module EM 34x (Part No. 72575P1 / P2). To connect the device (Fig. 4), use a communication cable of type I-Y (St) Y n x 2 x 0,8 mm or a comparable type.

After installation of the call module, it is essential to perform the addressing on the BBUS and to create a new database before the proper functioning can be ensured. See the documentation for additional information (Part No. 89440C4).

Additional and updated information

The product specifications given here relate to those at the time of printing and may differ due to subsequent modifications and / or amended Standards and Regulations. For updated information and declarations of conformity refer to www.ackermann-clino.de.

Refer to the System Catalogue for additional accessories.

The 6-pin socket of the call module (Part No. 73073D3) is not electrically isolated. Connected devices must have their own electrical isolation according to DIN EN 60601-1-1.

When an auto-release plug system or another call button is connected via terminals 4, 5, 6, 7, it is not possible to simultaneously use the 7-pin socket of the call module (Part No. 73073D3 / E3). To prevent simultaneous use, close the 7-pin socket with a dummy plug or the like.

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© 2010 Honeywell International Inc.